EN Dr. Brown's® Zero-Resistance™ Specialty Feeding System Includes the Infant-Paced Feeding Valve

Dr. Brown's® Zero-Resistance™ Specialty Feeding System was designed in collaboration with medical professionals seeking an oral feeding solution for infants identified with complex feeding challenges such as those often observed in infants with cleft lip and palate, ankyloglossia, high-arched palate, oro-neuromotor dysfunction, common or rare syndromic sequences and/or craniofacial anomalies.

The *Zero-Resistance™ Specialty Feeding System* assists infants who are unable to generate adequate suction to retrieve fluid from a bottle. This bottle system allows the infant to retrieve fluid by compressing the nipple/teat. The Infant-Paced Feeding Valve (blue one-way valve) can be inserted into any Dr. Brown's narrow nipple/teat to create a "compression" nipple/teat.

Benefits of the Zero-Resistance™ Specialty Feeding System:

- Consistent and reliable nipple/teat flow rates
- Infant directed feeding vs. manually expressed boluses by feeder
- Requires minimal assembly
- Helps decrease gassiness/air intake and burping with non-squeeze method
- Preserves Vitamins A, C, and E in breastmilk.
- Easy to clean and safe for dishwasher, sanitizer, and autoclave*
- Fully reusable system
- The full bottle system and nipples/teats are BPA free and do not contain natural rubber latex.
- Exhibits a typical feeding bottle design

***Autoclave (Steam Sterilization) is not to exceed 30 cycles.**

NOTE: With the Infant-Paced Feeding Valve (blue one-way valve) in place, the nipple/teat will fill with fluid as infant elevates tongue to the underside of the nipple/teat during feeding. No manual priming of the nipple/teat is necessary.

Technical Notes:

NOTE: The Zero-Resistance™ Specialty Feeding System WILL NOT FUNCTION as intended without the INSERT, RESERVOIR (Internal Vent System) and INFANT-PACED FEEDING VALVE. If the baby needs the Infant-Paced Feeding valve (blue-one way valve), leave the internal vent system in place.

- The *Zero-Resistance™ Specialty Feeding System* is composed of either a 4 oz/120 mL or an 8 oz/250 mL bottle system with the internal vent system, a silicone nipple/teat and the Infant-Paced Feeding Valve.
- Eight (8) different nipple/teat flow rates are available to custom-fit all infants' needs.
- This bottle system is not intended to be a disposable system. Prior to first use, wash with soap and water. In hospital/clinical setting, consider sanitizing. In home environment, boil for 5 minutes, rinse well, and air dry. Rinse immediately after use and clean with soap and water, rinse well, and air dry before reusing. In hospital/clinical environment, consider sanitizing once every 24 hours.
- Do not warm the liquid beyond body temperature of 98.6°F (37°C).
- If you are having difficulty using the *Zero-Resistance™ Specialty Feeding System*, call **Dr. Brown's Customer Service at 1-800-778-9001** or contact us at: **help@drbrownsbaby.com**. For purchases made outside the United States and Canada, please contact your local distributor.
- For more information about the *Zero-Resistance™ Specialty Feeding System* visit: **drbrownsmmedical.com/products/specialty-feeding**

Read and save these instructions for future use.

BEFORE FIRST AND EACH USE

Before first use, fully disassemble all parts, wash the product, then place the components in boiling water for five minutes. This is to ensure hygiene. To prevent a possible choking hazard, pull on the nipple/teat in all directions before each use to ensure it does not tear. Inspect carefully, especially when the child develops teeth. Discontinue use and replace the nipple/teat immediately if any signs of wear or damage are visible, such as cracking or change of shape. Small pieces can accidentally be inhaled and cause serious injury.

CLEANING

CAUTION: Do not use the wire cleaning brush to clean the nipple/teat as this will cause it to tear. Be sure to remove wire cleaning brush from bottle before cleaning. Fully disassemble all parts and wash prior to each use. All parts are safe for use in dishwasher (top rack only), electric sterilizer, microwave sterilizer, or boiling water. Do not microwave wire cleaning brush.

ASSEMBLY AND FEEDING

Please refer to diagram.

To assemble the internal vent system, connect the reservoir tube to the insert so that they fit tightly together. Place the assembled vent system into the bottle. Snap the nipple/teat into the collar, affix collar to bottle and turn until closed (do not over-tighten). If warming the bottle – always remove all bottle components before warming and use the storage/travel cap. Always test the temperature of milk before feeding baby.

Do not shake the bottle. Shaking will cause air bubbles. Formula should be stirred gently to fully dissolve lumps as lumps can clog nipple/teat holes. Fill to the desired amount. To prevent potential leaks, do not fill the bottle above the FILL LINE WARNING.

Options+ Bottles – With or Without Vent

It's recommended that the internal vent system be installed for the most comfortable feeding, however the vent system can be removed as baby's feeding develops; providing an experience similar to most nipple/teat vented bottles.

REMINDERS

For proper bottle and venting function, only use Dr. Brown's® nipples/teats with this bottle; other brands may leak or flow too fast or too slow. When feeding your baby, hold the bottle so that the reservoir tube hole in the bottom of the bottle is not covered by liquid (approximately at a 45° angle). Keep your baby propped up during feeding. Remove from your baby's mouth when they are not feeding. Change nipples/teats after six weeks of use.

When transporting bottles, always keep them upright and use storage/travel cap or disk to help prevent leaking. During air travel, remove internal parts and secure with travel cap or disk to prevent leaking from air pressure.


For your child's safety and health

WARNING!

- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Throw away at the first signs of damage or weakness.
- Keep components not in use out of the reach of children.
- Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
- Never use feeding nipples/teats as a pacifier/soother.
- Always use this product with adult supervision.
- Glass containers may break.
- Always check the nipples/teats for signs of wear and replace when damaged.
- Do not use a microwave oven to heat the bottle as microwave ovens can create hotspots in milk/formula and/or bottle components.
- Do not leave the nipple/teat in direct sunlight, heat, or in sterilizing solution for longer than recommended as this may weaken the nipple/teat.
- Do not allow a child to walk, or run while using any bottle.
- Do not let a child take a bottle to bed or self-feed for long periods.

GLASS WARNING

Dr. Brown's glass bottles are made of borosilicate glass. Glass bottles may break. Bottles should always be inspected before each use for sharp edges, fragments, or flakes of glass that may have become dislodged and loose inside the bottle. Do not use if there are any cracks or chips on the bottle. Sudden temperature changes can cause glass bottles to crack or break. Do not freeze glass bottles. **Never allow the child to stand, walk, or run while using a glass bottle. Do not allow baby to self feed.**

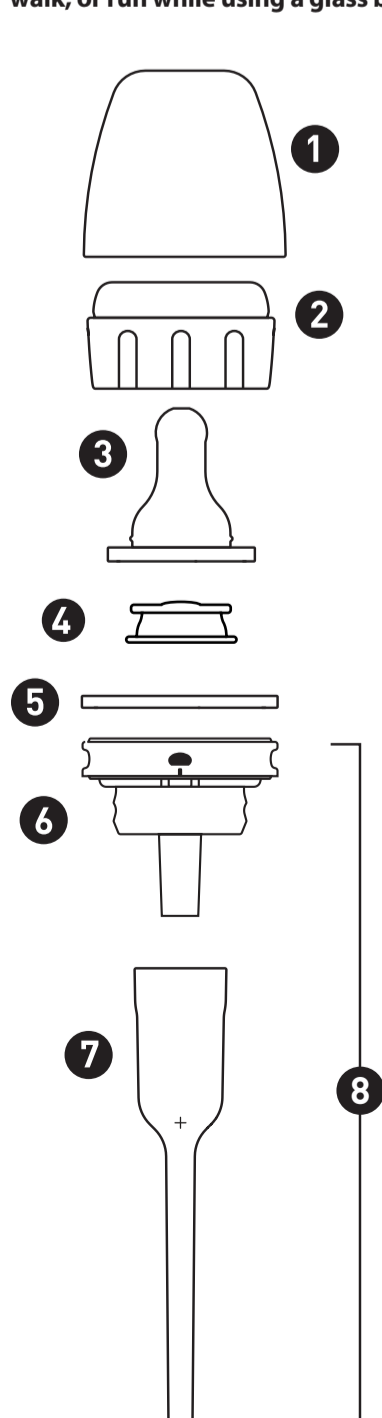Bottle Components

- 1 Cover/Cap
- 2 Collar
- 3 Nipple/Teat
- 4 Infant-Paced Feeding Valve (blue one-way valve)
- 5 Travel Disk
- 6 Insert
- 7 Reservoir Tube
- 8 **Vent Assembly:** Press firmly together to prevent leakage

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